


## COT Test Device Procedure Card

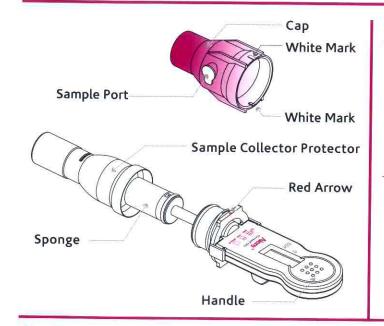


## Sample Collection

· Instruct the donor not to place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.



At least 10 minutes

· Remove Sample Collector Protector.

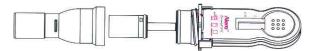



- · Insert Sponge end of collector into mouth.
- · Actively swab the inside of the mouth and tongue with Sponge. As soon as the Sponge softens slightly, gently press the Sponge between the tongue and teeth to ensure complete saturation. The Sponge is saturated when no hard spots can be felt. Collect for at least three (3) minutes before removing
- · Hand the saturated collector over to the tester.







3 minutes


## **Testing**

- A. Insert the collector vertically into the Cap and press down firmly until the Cap reaches the thread.
  - B. Twist the Handle clockwise 180° to tighten the Cap until the Red Arrow lines up with the other White Mark.
  - C. Place the test device horizontally on a clean and level surface with the result window facing up.

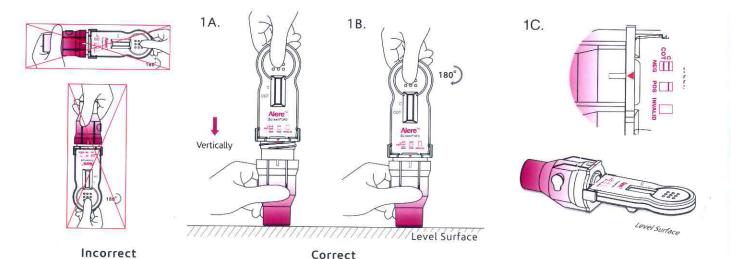

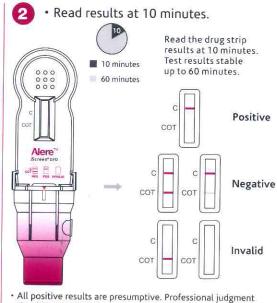

 All positive results are presumptive. Professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are indicated.

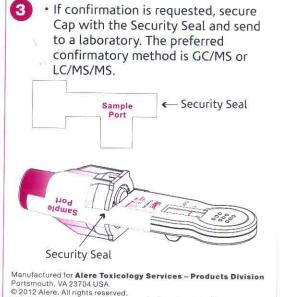

The Alere Logo, Alere and iScreen are trademarks of the Alere

group of companies

No.:1156050303