icaleri

NicAlert

6

5

3

2

1

DRUG TESTING KITS

- Detects recent tobacco use
- Use with saliva or urine
- · Results in minutes
- Semi-quantitative results

FLCO DANG AND ALCOHOL TESTING PRODUCTS

1-800-227-9890

www.alcopro.com

NicAlert

Detects Recent Tobacco Use

The NicAlert device is ideal for determining tobacco use. The NicAlert detects cotinine, the major metabolite of nicotine. Cotinine is the preferred marker for tobacco use because cotinine stays in the body much longer than does nicotine. The presence of cotinine in saliva or urine is a reliable indicator for tobacco use within the past several days.

Test Saliva or Urine

The NicAlert can be used to test urine or saliva. Both mediums provide reliable methods for determining tobacco use. Because most users prefer the convenience of saliva, we provide saliva collection containers with every kit. We include instructions and interpretation charts for using the NicAlert with saliva and with urine.

Semi-Quantitative Results

The NicAlert is the only nicotine device we know of that gives semi-quantitative results. The presence of nicotine in saliva or urine creates a red colored band on the strip. Higher concentrations of nicotine cause the band to move higher on the strip. The test strip is divided into six zones to assist the operator in interpreting test results. High concentrations of nicotine indicate recent tobacco use or heavy, chronic use.

SPECIFICATIONS

Test Medium: Saliva or Urine

Format: Dip stick

Measurement range: 0 - >2000 ng/ml cotinine



Reading = Level "4"
Positive Result

Reading = Level "0"
Negative Result

NicAlert Saliva/Urine Test

Item	Description	Price
122	NicAlert:Saliva Nicotine Test	\$14.00@
*Price subject to change. Call to confirm current pricing.		

For Information & Orders Call:
Order 24 Hours A Day At:
Fax Orders To:

1-800-227-9890 www.alcopro.com 1-800-655-5834

