


P.A.S. Sentry

P.A.S. Sentry

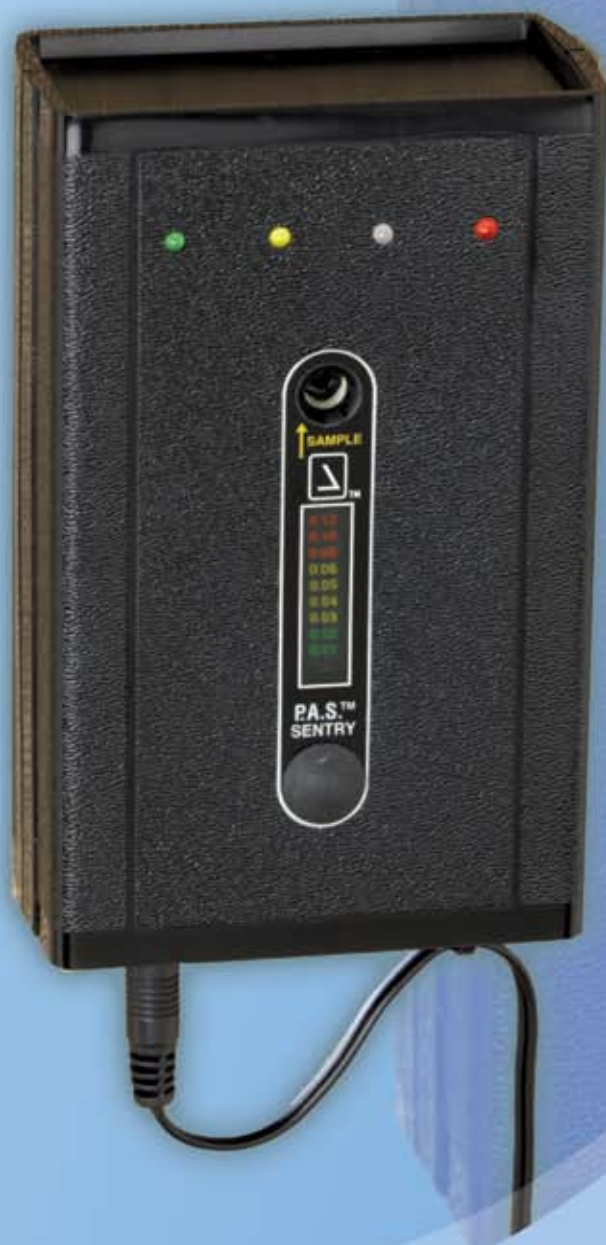

ALCOHOL TESTING INSTRUMENTS

- Passive testing without mouthpiece
- Optional wall mount
- Fuel cell sensor is specific for alcohol
- Zero operating cost

ALCO PRO
DRUG AND ALCOHOL TESTING PRODUCTS

1-800-227-9890

www.alcopro.com

P.A.S. Sentry

Passive Testing without a Mouthpiece

The PAS Sentry uses passive alcohol test technology to “sniff” the breath of a person for the presence of alcohol. To perform a test the operator presses the Start button and the subject blows at the PAS Sentry from a distance of several inches. A fan inside the device pulls in a portion of the subject’s breath to analyze for alcohol. Because the device does not use mouthpieces or batteries there is no operating cost to use the PAS Sentry.

Light Bar Indicates Presence of Alcohol

The PAS Sentry uses a series of colored lights to indicate the presence and relative amount of alcohol on a person’s breath. A person with no alcohol on their breath lights only the first green light. If a person’s breath contains alcohol a series of yellow and red lights turn on to indicate the presence and approximate amount of alcohol in the breath.

Fuel Cell Sensor Specific for Alcohol

Fuel cell sensors are typically used in top-of-the-line evidential breath alcohol testers because of their ability to detect low amounts of alcohol and because they are specific to alcohol. That means that operators using the PAS Sentry can have a high degree of confidence that positive results with the PAS Sentry are due only to alcohol.

Wall Mount for Easy Access



The PAS Sentry easily mounts on a wall so that it is readily accessible for testing. This feature is particularly useful in settings such as shelters and halfway houses that routinely test clients every time they enter the facility. The Velcro mounting tape included with the kit makes it easy for staff to remove the PAS Sentry from the wall to store the unit when not in use.



SPECIFICATIONS

Sensor type: Electrochemical Oxidation (fuel cell)
Results: Colored light bar
Breath sampling: Breath sample intake fan, manual start
Response time: 5 – 10 seconds negative results, 10 – 30 seconds positive results
Power requirement: 110 volt AC power (Power adapter included)
Test mode: Passive
Calibration: Using wet bath simulator
Temperature range: 10 – 40 °C
Size: 5 ¼" x 3 ¼" x 1 ¾", 8 oz.
Warranty: one-year parts and labor, two years fuel cell

RELATED PRODUCTS

Item	Description	Price
	220 P.A.S. Vr. <i>Includes battery, manual, 10 mouthpieces, and a carrying case.</i>	\$459.00
	218 P.A.S. IV <i>Includes rechargeable battery, AC charger, DC auto adapter, padded carrying case and operation manual.</i>	\$749.00

P.A.S. Sentry

Item	Description	Price
219	P.A.S. Sentry <i>Includes: Mounting Velcro tape, AC power supply, and manual.</i>	\$598.00

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: **www.alcopro.com**
 Fax Orders To: **1-800-655-5834**

ALCO PRO
 DRUG AND ALCOHOL TESTING PRODUCTS