RAPID TOX Rapid Tox Cup II CUP II



DRUG TESTING KITS

- Single or up to 14 panel configurations plus adulteration strips available
- Results ready in 5 minutes and stable up to 4 hours
- Female-friendly collection container
- Temperature strip verifies valid specimen
- Most economical collection and testing cup

ALCOPRO DRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890 www.alcopro.com

DRUG TESTING KITS

Rapid Tox Cup II



The Rapid Tox Cup II is a cost effective collection and testing cup, allowing you to stretch your drug testing budget.

Convenient Integrated Cup Design

The Rapid Tox Cup II minimizes contact with the urine by placing the drug detection strips inside the specimen container. Urine comes into contact with the strips and results are easy to read through the clear cup.

Easy to Read Results in Minutes

A valid test result produces a control line at the top of each strip to show that the test reagents are working correctly. Negative results may be interpreted once the control lines have formed and the background has cleared. The absence of a line at a drug indicator is read as a presumptive positive result. Read positive results in 5 minutes. **Results are stable for up to 4 hours.**

Female-Friendly Collection Container

Unlike other test kits that minimize cost by shrinking the container size, the Rapid Tox Cup II has a wider design, which makes voiding easier and more sanitary particularly for female test subjects.

Detects Up to 14 Drugs of Abuse

The Rapid Tox Cup II offers configurations that can test for a single drug or up to 14 drugs with a single test, so you can stretch your budgets. Multiple or single drug testing configurations provide you greater flexibility in your testing program.

SPECIFICATIONS	
Test Medium: Urine Format: Integrated Cup with Temperature Strip Cut-off levels:	
AMP (amphetamine)	1000 ng/ml
BAR (barbiturates)	300 ng/ml
BZO (benzodiazepines)	300 ng/ml
COC (cocaine)	300 ng/ml
MDMA (Ecstasy)	500 ng/ml
THC (marijuana)	50 ng/ml
MTD (methadone)	300 ng/ml
mAMP (methamphetamine)	1000 ng/ml
OPI (opiates)	300 ng/ml
OXY (oxycodone)	100 ng/ml
PCP (phencyclidine)	25 ng/ml
TCA (notriptyline)	10 ng/ml



EASE OF USE



Step 1 Check temperature range for possible adulteration.



Step 2
Read drug test results at five minutes.

RAPID TOX CUP II

ltem	Description	I - 24	Box of 25 Tests
8RT42	THC/coc/opi/mAMP	\$5.55	\$118.75 (4.75 ea)
8RT51	THC/coc/opi/amp/pcp	\$5.95	\$128.75 (5.15 ea)
8RT61	THC/coc/opi,mAMP/amp/pcp	\$6.65	\$146.25 (5.85 ea)
8RT81	THC/coc/opi/mAMP/mdma/ mtd/oxy/bup	\$9.05	\$211.25 (8.45 ea)
8RT101	THC/coc/opi/mAMP/amp/ bar/mtd/pcp/bzo/ppx	\$9.75	\$223.75 (8.95 ea)

For Information & Orders Call: 1-800-227-9890
Order 24 Hours A Day At: www.alcopro.com
Fax Orders To: 1-800-655-5834

