CEY CUP

Split Specimen Key Cup



DRUG TESTING KITS

- Test card integrated into collection container
- Operator activates test with special key
- Split specimen design segregates specimen from test card
- Flat front design enables photocopying of test results

ALCOPRODRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890 www.alcopro.com

DRUG TESTING KITS

Split Specimen Key Cup



The Split Specimen Key Cup uses an integrated cup design that builds a drug detection card securely into a urine specimen container to minimize the collector's contact with the urine specimen. The donor provides a urine specimen directly into the cup and attaches the leak proof screw-on lid. The integrated cup design eliminates the need to pipette the urine onto a test card or to dip a test strip into the urine.

Operator Controls Test Activation

The unique design of the Split Specimen Key Cup allows only the operator to activate the test device by inserting a special to start the test reaction. Other cups start to react immediately, as soon as the donor's specimen fills the cup. This important feature places the test procedure entirely under the control of the operator.

Photocopy Test Results

The flat front design of the Split Specimen Key Cup enables the operator to photocopy test results to create a permanent record. We include a photocopy report template so that collectors can document donor identification and other relevant details of the collection process as well as the test results.

Secure Laboratory Confirmation

The design of the Split Specimen Key Cup keeps the urine specimen segregated from the test card at all times. This provides the highest level of security when sending specimens to the laboratory for confirmation testing. The operator sends the specimen in the original cup without the need to pour the specimen into another container.

SPECIFICATIONS

Test Medium: Urine

Format: Integrated split specimen cup with temperature strip

Cut-off levels:

Cut-Oil levels.	
AMP (amphetamine)	1,000 ng/ml
BAR (barbiturates)	300 ng/ml
BZO (benzodiazepines)	300 ng/ml
COC (cocaine)	300 ng/ml
MDMA (Ecstasy)	500 ng/ml
THC (marijuana)	50 ng/ml
MTD (methadone)	300 ng/ml
mAMP (methamphetamine)	1,000 ng/ml
MOR (morphine)	300 ng/ml
OPI (opiates)	2,000 ng/ml
OXY (oxycodone)	100 ng/ml
PCP (phencyclidine)	25 ng/ml
PPX (propoxyphene)	300 ng/ml



EASE OF USE



Step 1

The donor provides the specimen with the lid securely fastened.



Step 2

The operator inserts the key.



Step 3

Pressing the key allows a small portion of the specimen to contact the test card, activating the test.



Step 4

The presence of a line at the drug indicator shows negative results for that drug. The control line indicates the test is working properly.

Split Specimen Key Cup

ltem	Description	1-24	Box of 25
8K41*	THC/coc/mAMP/opi	\$6.40	\$135.00 (\$5.40 each)
8K53*	THC/coc/mAMP/opi/amp	\$6.60	\$137.50 (\$5.50 each)
8K61	THC/coc/mAMP/opi/amp/bzo	\$7.30	\$155.00 (\$6.20 each)
8K101	THC/coc/mAMP/opi/amp/bzo/ pcp/bar/mtd/oxy	\$11.25	\$248.75 (\$9.95 each)