
Procedure for Using Foil

- 1. Place ROOT END at the EDGE of the foil where it is marked "Place ROOT END HERE"
- 2. Straighten hair to full length along center crease of foil keeping the root ends together at the edge of the foil.
- 3. Fold foil down center line
- 4. Fold again LENGTHWISE
- 5. Wrap any excess hair extending beyond foil around the foil







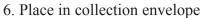



FINISHED

NOTE - Please do not crumple the foil!





Omega Laboratories, Inc. * 400 N. Cleveland Avenue * Mogadore, Ohio * 44260 www.omegalabs.net * 800.665.5569