

SWEAT PATCH

Sweat Patch Drug Test System



SWEAT PATCH FEATURES

- Continuous monitoring for seven days
- Patch stays on through showers, swimming, workouts
- Resists tampering and adulteration
- Includes laboratory GC/MS confirmation of positive results
- For treatment and drug diversion programs

ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890
www.alcopro.com

Sweat Patch Drug Testing System



Continuous Monitoring for Seven Days

The Sweat Patch drug test system is designed to be worn for seven days and tests for the 5 common drugs of abuse (marijuana or THC, cocaine, opiates, amphetamines and PCP). The sweat patch consists of a collector pad held in place by a strong adhesive film. If the subject uses drugs while wearing the patch, a small portion of the drug and/or drug metabolite is excreted into the sweat and absorbed into the collector pad. The sweat patch provides continuous monitoring the entire time the subject wears the patch. It is water resistant, so it stays on during showers and swimming.

Resists Tampering and Adulteration

None of the common methods used to defeat urine drug testing are effective with the sweat patch. Drinking fluids and ingesting cleansers have no effect on the sweat patch. High Times magazine gave the sweat patch an unintentional endorsement when it advised readers to avoid wearing the patch if possible.

Includes Laboratory GC/MS Confirmation Testing

All sweat patch drug tests are analyzed by a certified laboratory. Cut-off levels correspond to levels of detection in urine. Non-negative screening results are confirmed using the gold-standard GC/MS methodology to produce evidential-quality results. The sweat patch has a long history of successful defenses against court challenges.

Sweat Patch Drug Test System

Item	Description	Price
882	Sweat Patch Drug Test (screens for THC, cocaine, opiates, PCP amphetamines, methamphetamines)	\$49.00
882-Opiate	Sweat Patch + Extended Opiate Panel (screens for THC, cocaine, opiates, PCP amphetamines, methamphetamines, hydrocodone, hydromorphone, oxycodone, oxymorphone)	\$87.00

Related Items

Item	Description	Price
882-Overlay	Sweat Patch Overlay	\$ 1.25 ea
857	Gloves, Latex (sm, md, lg)	see website

Specifications

Test Medium:	Sweat										
Format:	Absorbent Collector Pad										
Cut-off Levels:	<table border="0"> <tr> <td>THC</td> <td>0.5 ng/ml</td> </tr> <tr> <td>cocaine</td> <td>10 ng/ml</td> </tr> <tr> <td>opiates</td> <td>10 ng/ml</td> </tr> <tr> <td>amphetamine</td> <td>10 ng/ml</td> </tr> <tr> <td>PCP</td> <td>7.5 ng/ml</td> </tr> </table>	THC	0.5 ng/ml	cocaine	10 ng/ml	opiates	10 ng/ml	amphetamine	10 ng/ml	PCP	7.5 ng/ml
THC	0.5 ng/ml										
cocaine	10 ng/ml										
opiates	10 ng/ml										
amphetamine	10 ng/ml										
PCP	7.5 ng/ml										

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: www.alcopro.com
 Fax Orders To: **1-800-655-5834**

ALCOPRO
 DRUG & ALCOHOL TESTING PRODUCTS