



CheckPoint


Breath Alcohol Test

Pass/Fail Results Instantly.

The CheckPoint breath alcohol test is an easy-to-use crush and blow alcohol test that provides results in only two minutes. The CheckPoint comes in three cut-off levels and is an ideal alcohol screening device where zero tolerance alcohol programs need to be enforced.

The subject squeezes the tube and blows into the end of the tube for 12 seconds. The yellow crystals turn color when alcohol is detected at the cut-off level.

- Able to detect the presence of alcohol at 0.02%, 0.04% and 0.08% BrAC cut-off levels
- FDA approved
- Easy to use crush and blow screening test
- Disposable and cost-effective test device
- No operator training or certification is required
- Provides Pass/Fail results in only two minutes
- Ideal for non-regulated workplace testing, corrections, military, substance abuse or other markets where safety and compliance are important

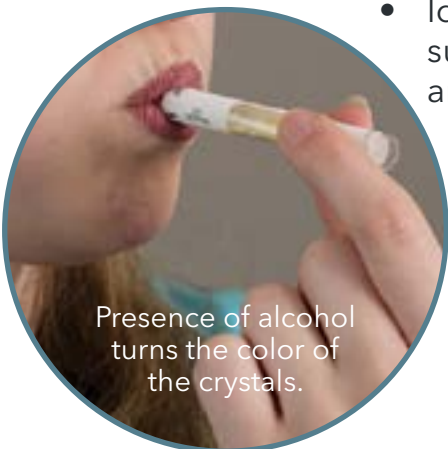

Presence of alcohol turns the color of the crystals.

ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS

SPECIFICATIONS

Test Medium:	Breath	Test Principle:	Dichromate chemical
Reading Time:	Two minutes	Results:	Qualitative Pass/Fail
Cut-Off Levels:	0.02%, 0.04%, 0.08% BrAc	Shelf Life:	Three years from manufacture date
FDA Approved:	Yes		

ITEM	DESCRIPTION	1-49	50-99	100+
CCP02200	0.020% cut-off threshold	\$2.95	\$2.75	\$2.55
CCP04200	0.040% cut-off threshold	\$2.95	\$2.75	\$2.55
CCP08200	0.080% cut-off threshold	\$2.95	\$2.75	\$2.55

EASE OF USE



Step 1
Crush the test ampule between your fingers.



Step 2
Have the subject blow into the end of the test ampule for 12 seconds.



Step 3
At 2-minutes, observe the color in the test ampule. A blue or green color indicates the presence of alcohol at the designated cut-off level.



A Few Good Reasons To Trust Us



Same day shipping
Most orders placed before 2:00 pm EST ship same day.



Knowledgeable Customer Support
Chat online or by phone.



Trusted Supplier more than 40 years.
Supporting our customers since 1982.



Largest inventory
We stock more, so you never run out.