



Q.E.D.

Saliva Alcohol Test

DOT Approved Alcohol Screening Device.

The Q.E.D. is approved by the U.S. Department of Transportation as an Alcohol Screening Device (ASD), and it appears on the NHTSA's Conforming Products List (CPL).

Because the proportion of alcohol in saliva is directly related to the proportion of alcohol in the blood, the Q.E.D. is able to quickly and accurately measure intoxication levels. When alcohol is present in the saliva, the Q.E.D. displays a column of reddish-purple color in the test column.

- Semi-quantitative results
- Fast and easy to use with results in only two minutes
- Only a small amount of saliva is required to activate the test
- Ideal for DOT testing programs
- Detects alcohol in saliva as low as 0.010% up to 0.15% BAC
- CLIA Waived for professional use



ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS

SPECIFICATIONS

Test Medium: Saliva

Test Medium Enzyme

Reading Time: Two minutes

Results: Semi-quantitative

Shelf Life: Twelve months from manufacture date

Certifications: CLIA Waived

Easy To Use



Step 1

Saturate the test pad with saliva for 60 seconds.



Step 2

Press the saturated swab into the test well. The capillary action draws the saliva up the column. At two minutes, read the results.

Like reading a thermometer, the operator reads the results by looking at the upper limit of the color in the test column.

DOT Training & Certification



Our Screening Test Technician training complies with Federal regulations and includes device proficiency training. Train at your pace with our highly interactive training course.

A Few Good Reasons To Trust Us



Same day shipping

Most orders placed before 2:00 pm EST ship same day.



Knowledgeable Customer Support

Chat online or by phone.



Trusted Supplier more than 40 years.

Supporting our customers since 1982.



Largest inventory

We stock more, so you never run out.