

Alco-Sensor III

Alco-Sensor III



%BAL

0.021

ALCO-SENSOR III

INTOXIMETERS, INC.
SAINT LOUIS, MISSOURI

ALCOHOL TESTING INSTRUMENTS

- Displays results to three decimal places
- Evidential accuracy
- Fuel cell sensor is specific for alcohol
- Easy two-button operation
- Mouthpiece ejector button

ALCO
DRUG AND ALCOHOL TESTING PRODUCTS

1-800-227-9890

www.alcopro.com

Alco-Sensor III

Evidential Accuracy

The Alco-Sensor III is approved by U.S. Dept. of Transportation as an evidential breath testing device. The Alco-Sensor III is approved for use by law enforcement agencies in almost every state as a PBT (Preliminary Breath Tester). Accurate to +/- .005 BAC.

Easy Two-button Operation

Two large buttons on the front of the Alco-Sensor III control the operation of the instrument. The operator presses the SET button to prepare the instrument for a test. As the subject blows through the instrument the operator presses the READ button to capture a sample of the subject's breath for analysis. The Alco-Sensor III displays results within seconds. The display brightens to indicate the final result which can be displayed by pressing the READ button again.

Display Messages Guide Operator

The Alco-Sensor III prompts the operator to perform several key functions. For example, the Alco-Sensor III displays "SET" after each test to prompt the operator to press the SET button. The instrument displays "RCL" (abbreviation for Recall) when it displays the last test result. "CAL" prompts the operator to perform the Calibration procedure.

Secure Calibration Procedure

The Alco-Sensor III now uses a secure calibration procedure that eliminates accidental tampering with the calibration of the instrument. The operator must place the instrument into calibration mode before adjusting the instrument. This new calibration procedure safeguards against unauthorized calibration adjustments.



SPECIFICATIONS

Sensor type:	Electrochemical Oxidation (fuel-cell)
Range of measurement:	Accurate between .000 - .400 BrAC
Accuracy and precision:	U.S. DOT approved for evidential use
Breath sampling:	Manual
Response time:	5 seconds negative results, 5 - 15 seconds positive results
Power requirement:	9-volt battery (included)
Test mode:	Direct
Calibration:	Microprocessor controlled, using ethanol, gas or wet bath simulator
Memory:	Last test
Print capability:	No
Temperature range:	0° - 40° C
Size:	2 1/2" x 4 1/2" , 6 oz.
Warranty:	One year, parts and labor

RELATED SUPPLIES

Item	Description	Price
305	Mouthpiece for Alco-Sensor	\$.24 @ 1-499 \$.23 @ 500-999 \$.22 @ 1,000-4,999 \$.20 @ 5,000-9,999 \$.17 @ 10,000+
303	Mini Alcohol Can	\$31.00
324	Alcohol Gas Tank, 105 liter	\$165.00
360	Alcohol Gas Tank, 30 liter	\$139.00
328	Regulator	\$109.00
313	Using the Alco-Sensor III DVD	\$50.00

Alco-Sensor III

Item	Description	Price
204	Alco-Sensor III	\$519.00

Includes hard-shell carrying case, 25 mouthpieces, mini alcohol can, battery, manual.

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: **www.alcopro.com**
 Fax Orders To: **1-800-655-5834**

ALCO PRO
 DRUG AND ALCOHOL TESTING PRODUCTS